


**Ankle Pain (Post-op)**

Quadripolar placement of the electrodes so the current crosses through the area of pain. Patient should feel the stimulation over the painful region.



**Interferential Quadripolar Diagram**

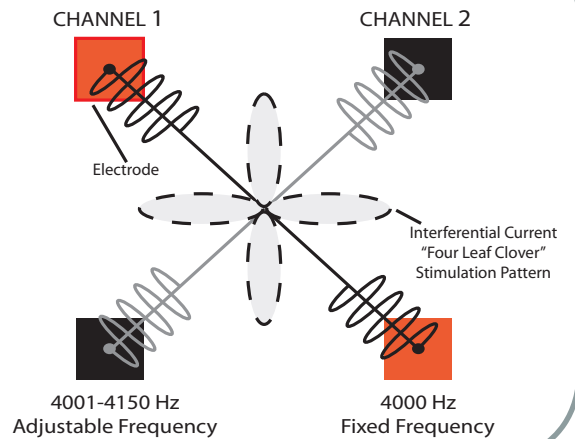

**Suggested Treatment Parameters**

**First Half of Treatment:**

- Beat Frequency: 150 Hz
- Amplitude: Strong but Comfortable (sensory level stimulation)
- Treatment Time: 20 minutes

**Second Half of Treatment:**

- Beat Frequency: 20 Hz
- Amplitude: Maximal Comfort Level (sensory to mild motor level stimulation)
- Treatment Time: 20 minutes

Sessions per day: 1-3

**Suggested Electrode Placement**

