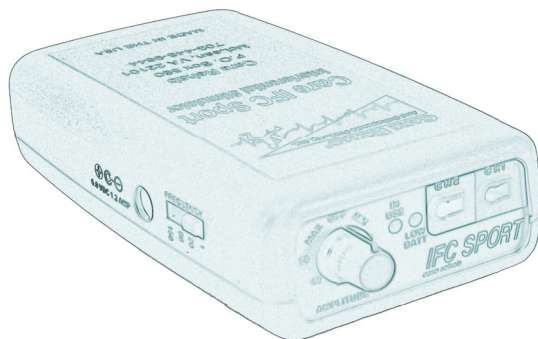
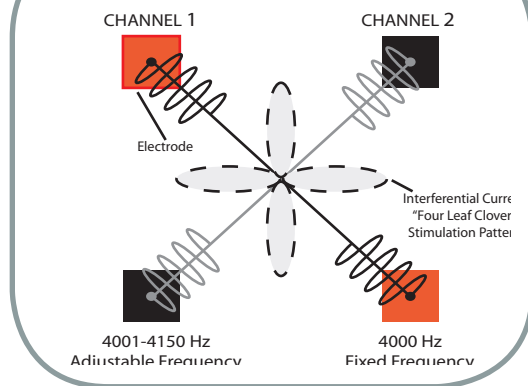
**Edema**

Swelling in the foot and/or ankle region secondary to trauma, surgery or venous insufficiency.

Interferential Quadripolar Diagram**Suggested Treatment Parameters****First Half of Treatment:**

- Beat Frequency: 20 Hz
- Amplitude: Strong but Comfortable (sensory level stimulation)
- Treatment Time: 20 minutes

Second Half of Treatment:

- Beat Frequency: 1 Hz
- Amplitude: Maximal Comfort Level (sensory to mild motor level stimulation)
- Treatment Time: 20 minutes

Sessions per day: 1-3

Suggested Electrode Placement