






Tarsal Tunnel Syndrome


Compression of the posterior tibial nerve (inside/medial aspect of the ankle), which may cause pain, burning and tingling in and around the ankle.

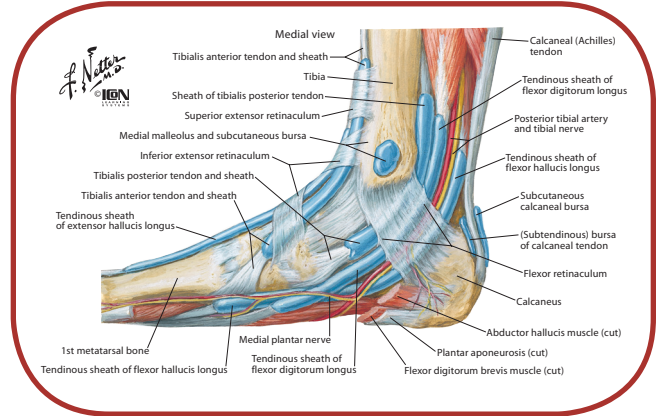

Interferential Quadripolar Diagram

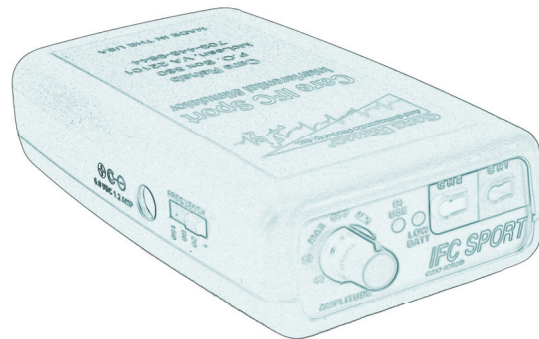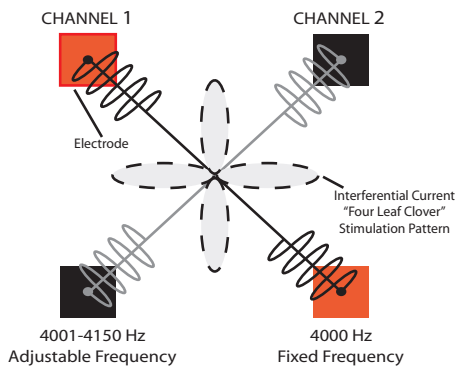

Suggested Electrode Placement

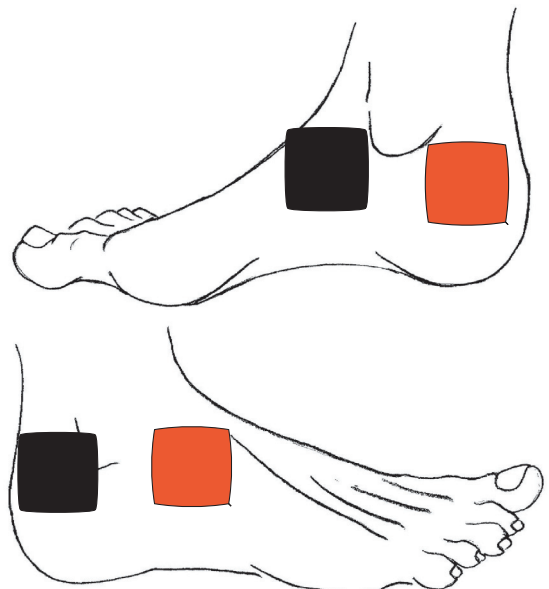

Suggested Treatment Parameters

First Half of Treatment:

- Beat Frequency: 150 Hz
- Amplitude: Strong but Comfortable (sensory level stimulation)
- Treatment Time: 20 minutes

Second Half of Treatment:

- Beat Frequency: 20 Hz
- Amplitude: Maximal Comfort Level (sensory to mild motor level stimulation)
- Treatment Time: 20 minutes

Sessions per day: 1-3