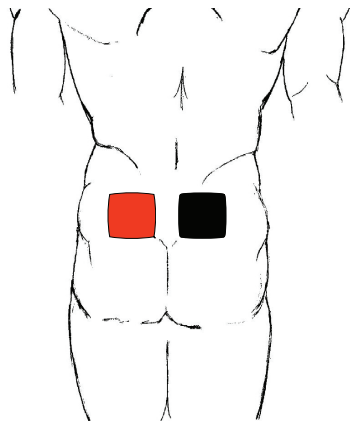


**Neuralgia (Lower Extremity)**


Neuralgia usually involves pain and radiculopathy within the lower leg. TENS is often used as an effective modality to reduce the patient's symptoms.

Suggested Electrode Placements**2-pad placement**

Place electrode pads over
bilateral lumbar spine
OR 1 pad on the lumbar
spine, 1 pad on buttocks
or posterior thigh

**4-pad placement**

Place electrode pads with 2
pads across bilateral lumbar
spine and 2 pads on but-
tocks or posterior thigh

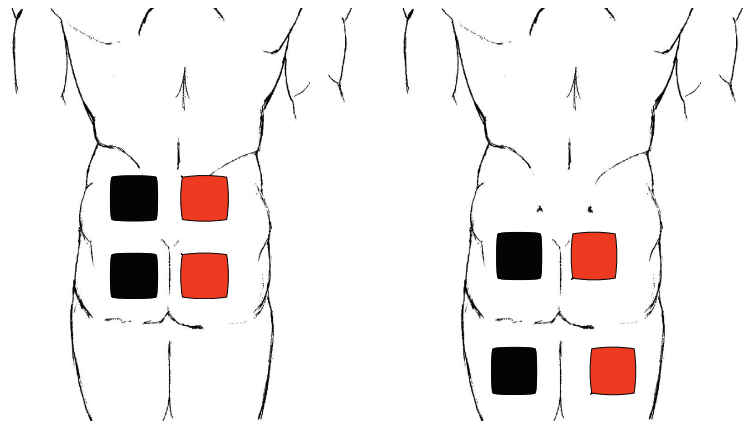**Suggested Treatment Parameters**

Mode: Modulated

Pulse rate: 100-120 Hz

Pulse width/duration: 50-150 μ s

Amplitude: Low to moderate level stimulation

Treatment Time: Throughout the day as needed

Mode: Continuous

Pulse rate: 80-100 Hz

Pulse width/duration: 50-100 μ s

Amplitude: Low to moderate level stimulation

Treatment Time: Throughout the day as needed

